Model: **pM-AD02**

TUBULAR ANTI-DECUBITUS MATTRESS SYSTEM WITH COVER & CPR
KILIFLI VE CPR ÖZELLİKLİ BORU TİPİ HAVALI ESKAR YATAK SİSTEMİ

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EN Instruction Manual

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EN Tubular Anti-Decubitus Mattress System With Cover & Cpr

Operating Principles

- Alternating pressure at a sufficient rhythm avoids prolonged vascular compression which is likely to cause tissue hypoxia.
- Alternating pressure adapts the mattress to the patient's and their position.

Guidelines for Use

The traditionally accepted method of pressure sore control is to turn the client at regular intervals and to observe the areas of risk regularly.

Alternating pressure mattress is intended to compliment this treatment, observation of the client and turning when necessary should still be considered as part of the treatment.

General

- For patients who are likely to spend more than 15 hours per day in bed.
- For patients who are classed as medium to high risk of developing pressure ulcers.

Installation/Use

1. Unroll the mattress.
2. Position don the bed.
3. Hang the compressor (pump) at the end of the bed (foot end) using the hooks.
4. Connect the mattress feed pipes to the compressor (pump) connector by matching the appropriate outlets.
5. Connect the electrical power supply cable to the compressor (pump).
6. Guide it along the bed to the mattress plug socket and plug in
7. Press switch to turn on the compressor (pump).
8. Before positioning the patient onto the mattress, wait until the mattress has been fully inflated.
9. Select the pressure corresponding to the patient's morphology.

Safety Measures

- DO TAKE CARE when operating this electrical equipment.
- DO NOT leave connected to electrical supply when not in use.
- DO NOT expose the equipment to excessive humidity.
- DO NOT use corrosive cleaning products such as industrial degreasers or acetic solvents.
- DO NOT attempt to relocate this equipment whilst connected to the electrical supply.

EN Cleaning / Disinfection

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- DO TAKE CARE to disconnect from the electrical supply before attempting to clean.
- DO TAKE CARE to use a cloth impregnated with a mild detergent solution.
- DO TAKE CARE to follow manufacturer's solution concentrations
- DO TAKE CARE not to exceed manufacturer's recommendations before wiping off

Maintenance

1. Compressor (pump) maintenance operations must be strictly carried out by technician authorized and trained by the importer.

EN CPR - Fast Air Release Valve

1. For emergent CPR, please pull the CPR yellow strip outside quickly to reduce the air fast At first the air under the chest part will be releasing fast and then the air under the head part and balance part will be releasing.
2. Normally close the CPR cover to reduce the air leakage.
3. When folding air mattress, open the CPR cover to release the air fast

Cover – Clean & Tidy Keeping

1. Cover can avoid bacteria and odor, it is waterproof.
2. Clean & tidy the cover every week even the cover is not dirty.
3. When the cover is dirty, please wash the cover by clean water. Put the cover onto the air mattress after dry.

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