

Specifications:

- It is used by fixing to the insulin pen.
- Sterile and disposable.
- Non toxic, non pyrogenic.
- Fits all insulin pens.

Instructions:

- Peel off the protective tab.
- Twist it in clockwise direction with the outer protective cover and fix to the pen.
- Take off the outer cover and save it.
- Take off the inner cover.
- Before injection ,check that the Fastfine pen needle allows free flow of medication ,following the instructions for use for your injection pen (priming test)
- Prickle to the skin and inject in concordance with your doctor's advice.
- Fix the outer cover carefully.
- After usage take off the needle by turning on anticlockwise direction and destroy it safely.

Cautions:

- Multiple usage of the same needles or to change the injection area irregularly may make sclerosis.
- To avoid damage injection areas should be controlled regularly.
- Don't use any needles if the protective tab is missing or damaged.

Storage: Store in cool and dry place.

Özellikler:

- İnsülin kalemine takılarak kullanılır.
- Sterildir ve tek kullanımlıktır.
- Toksik ve pirojenik madde içermez.
- Tüm insülin kalemleri ile uyumludur.

Kullanım talimatları:

- Koruyucu folyo etiketi çıkartın.
- İğneyi dış koruyucu kapağı ile saat yönünde çevirerek kaleme sabitleyin.
- Dış koruyucu kapağı çıkartın ve kapağı saklayın.
- İç koruyucu kapağı çıkartın.
- Enjeksiyondan önce, enjeksiyon kaleminizin kullanım talimatlarını takip ederek, Fastfine kalem iğnesinin ilaç akışına izin verdiğini kontrol edin(ilk test).
- Cilde batırın ve doktorunuzun tavsiyesine uygun olarak yavaşça enjekte edin.
- Dış koruyucu kapağı dikkatlice kapatın.
- Kullanımdan sonra kapağı saat yönünün tersine çevirerek iğneyi çıkartın ve güvenli bir şekilde atın.

Uyarılar:

- Aynı iğnenin çok defa kullanılması veya düzensiz enjeksiyon bölgesi değişimi doku sertleşmesine neden olabilir.
- Zarardan kaçınmak için enjeksiyon bölgelerinin düzenli olarak kontrol edilmesi gerekir.
- İğnenin koruyucu tabakası kalkmış ya da zarar görmüşse asla kullanmayınız.

Depolama: Serin ve kuru yerde saklayınız.

Trimpeks İth.İhr.Tur.ve Tic.A.Ş.

Eski Büyükdere Cad. Yunus Emre Sok.
Topçu İş Merkezi No:1/12,
34418, 4.Levent, İSTANBUL, TÜRKİYE
Tel +90 212 319 50 00
Fax +90 212 319 50 50



130 x 125 mm (Kapalı hali 65 x 125 mm)