## Ekran Mesajları

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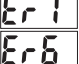
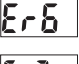
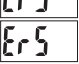
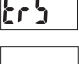
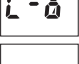
## Display Messages

	Meter detects blood glucose level <b>higher</b> than 600 mg/dL (33.3 mmol/L).
	Meter detects blood glucose level <b>lower</b> than 20 mg/dL (1.1 mmol/L).
	Ambient temperature was too high to perform the test.
	Ambient temperature was too low to perform the test.
	Test strip is not inserted properly, defective test strip period.
	
	Meter malfunction. Press "reset" button and check the meter again with the test strip. If the problem persists, contact technical service for help.
	
	Battery power is low. Meter will provide approximately 50 more measurements.
	Battery power is low. Replace immediately.

## Precautions for Use:

We are not responsible for any injury resulting from incorrect use of the plusMED fasTTest Monitoring System and not in line with the instructions set forth in this operating manual.

**Only use plusMED fasTTest Test Strip with the plusMED fasTTest Meter.**

Any change or administer of medication based on the plusMED fasTTest blood glucose test results without the consent and advice of a physician or healthcare professional is not recommended.

**plusmed**

BLOOD GLUCOSE MONITORING SYSTEM

**fasTTest**

**Quick  
Reference  
Guide**

CE 1783

To turn the Meter on, first insert the battery, then press the reset button located at the back of the Meter and proceed with the following instructions.

## Test Summary

### Step 1

Insert the Test Strip, white side face up, contact bar's end first, into the test port. A beep will sound and moving dotted lines will appear followed by a flashing "●" symbol. Now you are ready to obtain a blood sample.

### Step 2

Place blood sample onto the test well of the Test Strip. Wait until Confirmation Window has completely filled with blood and meter will beep.

### Step 3

After 5 seconds, your blood glucose result will appear on the display and will be stored into the Meter memory automatically.

## Memory Feature

### Step 1

Press the M button to turn on the Meter. The date and time will appear first. Each time you press the S button, the result of control solution, 7-day, 14-day, 30 days average will be shown.

### Step 2

Each time you press the S button, the Meter will recall the last 300 test results accordingly. When the memory is full, the oldest result is removed and replaced with the newest result.

### Step 3

To exit the memory mode, press and hold the M button to turn off the Meter.

### BEFORE TESTING:

Read detailed instructions in your plusMED fasTTest User's Manual

### Trimpeks İth.İhr.Tur.ve Tic.A.Ş.

Eski Büyükdere Cad. Yunus Emre Sok. Topçu İş Merkezi No:1/12,  
34418, 4.Levent, İSTANBUL, TÜRKİYE Tel +90 212 319 50 00 Fax +90 212 319 50 50