plusmed

health management

Model: **pM-FM**

FULL FACE MASK
YÜZ MASKESİ

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EN Instruction Manual

www.plusmed-health.com

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Thank you for choosing PLUSMED pM-FM Full Face Mask.

This mask is comfortable, flexible and quiet. It is designed in a way to minimize touch on your face and therefore make you feel comfortable during treatment. This user's manual provides information required for you to use the mask correctly.

Intended Use

PLUSMED pM-FM Full Face Mask directs the airflow coming from positive airway pressure device such as continuous airway pressure (CPAP) or bi-level systems in a non-invasive way to the patient. PLUSMED pM-FM Full Face Mask:

- Should be used by adult patients (> 30 kg) prescribed with positive airway pressure treatment.
- It is in a quality to be used by one patient in domestic atmosphere again, and used by more than one patient in hospital/health center environment.

CAUTION: Federal laws restrict the sale of this device only by doctors or based on doctor order.

Before Using the Mask


⚠ WARNINGS

- Ventilation holes must be open.
- **Warning Description:** CPAP systems are intended to be used with special masks with connectors which have vent holes to allow continuous flow of air out of the mask. When the CPAP machine is turned on and functioning properly, new air from the CPAP machine flushes the exhaled air out through the attached mask exhalation port. However, when the CPAP machine is not operating, enough fresh air will not be provided through the mask, and exhaled air will be rebreathed. Rebreathing of exhaled air for longer than several minutes can, in some circumstances, lead to suffocation. This warning applies to most models of CPAP systems.
- At low CPAP pressures the flow through the exhaled port may be inadequate to clear all exhaled gas from the tube. Some rebreathing may occur.
- To minimize the risk of vomiting during sleep, the patient should abstain from eating and drinking anything during the last 3 hours before using the mask. If the patient is using a prescriptive drug, it is suggested not to use this mask.
- Do not use the Mask if Rebreathing Prevention Valve is not attached.
- This mask should not be used on patients who are uncooperative, obtunded, unresponsive, or unable to remove the mask.
- When using oxygen as reinforcement, obey all the instructions.
- While flow generator is not working, oxygen flow must be cut and thus the risk of fire caused by oxygen accumulation in flow generator container should be prevented.
- In a constant oxygen flow rate, respired oxygen density varies based on pressure adjustments, patient's breathing pattern, mask, application point or leakage rate/amount.
- The technical specifications of the mask are provided for your clinician to check if it is compatible with the flow generator. If it is used beyond technical specifications

or used with incompatible devices, the seal and comfort of the mask may not be effective, optimum therapy may not be achieved, and leak, or variation in the rate of leak, may affect the function of the flow generator.

- If you notice ANY side effects while using the mask, please stop using PLUSMED pM-FM Full Face Mask and contact your physician or sleep therapist.
- To get detailed information about settings and usage information, check your flow generator user manual.


PLUSMED pM-FM Full Face Mask Parts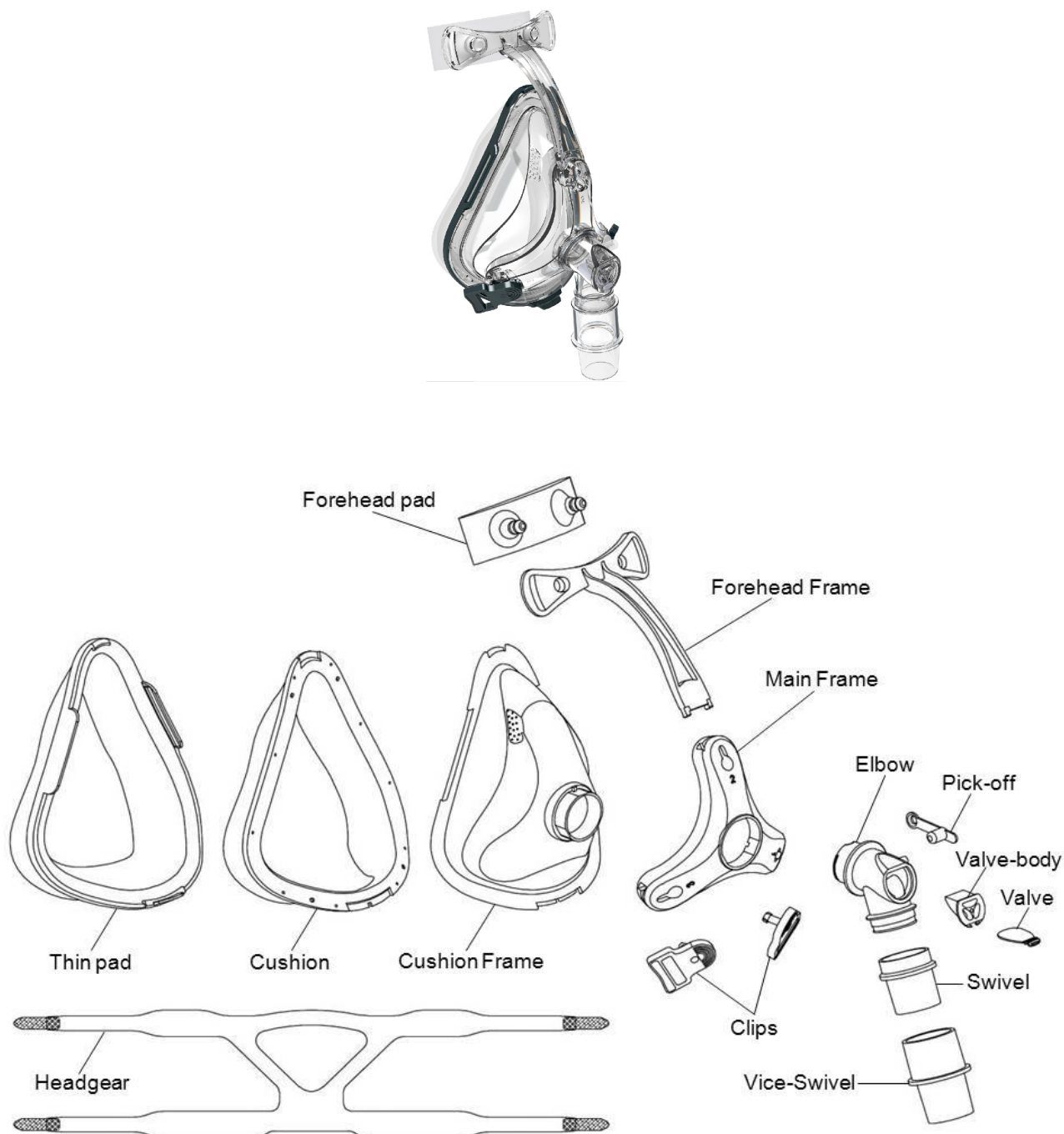

PLUSMED pM-FM Full Face Mask Parts

Finding The Right Buffer Size


- The following diagram depicts different facial features and face dimensions you need to measure (height, expressed with H).
- For a comfortable treatment, three different dimensions are provided for masks.
 - Choose the correct size by using the table below.

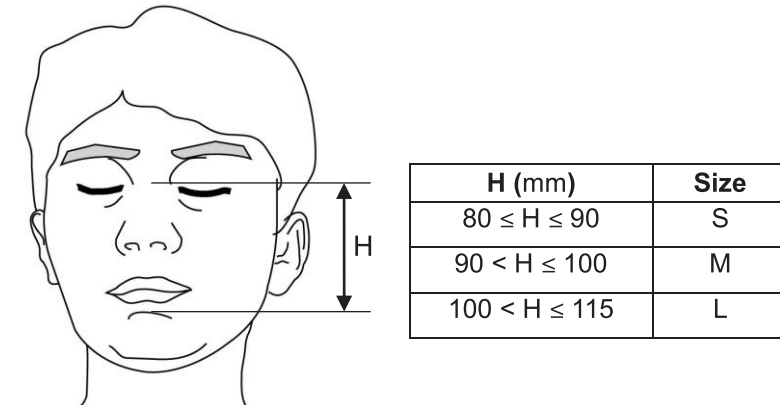Finding the Right Buffer Size


Fitting the Mask

1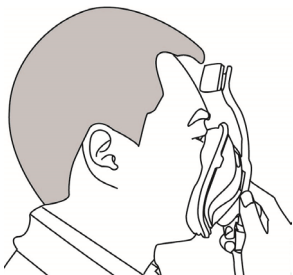


To determine whether the angle around the forehead suits you, hold the mask on your mask. If you think the angle is correct, skip to step 4.

2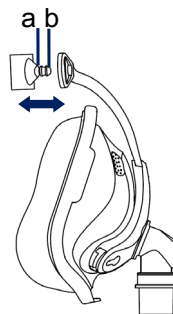


Set the forehead filling, by pushing or pulling the pawls in forehead filling. If it fits, proceed to the 4th step.

3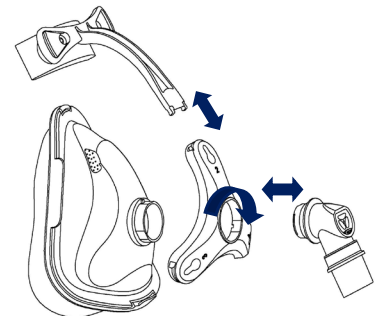

To adjust the forehead filling, check the section name "Adjusting the Forehead Filling". Repeat the previous steps. There are three levels defined as "1", "2" and "3".

4

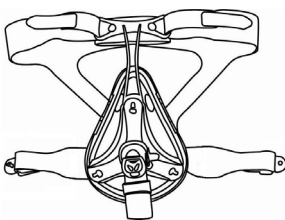
Attach the head apparatus by passing through holes and clips around the forehead frame.

5


Hold the mask to your face and pull the head apparatus belts to the back of your head.

6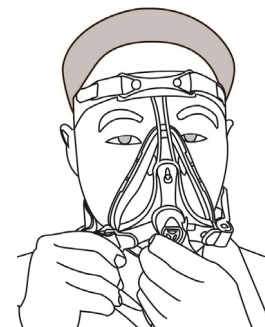


Place the sub belts under your ears and connect clips to the main frame.

7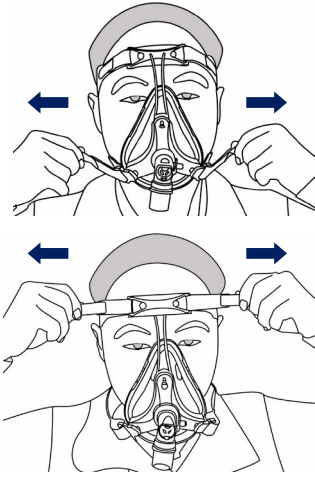

Equally adjust belts until it becomes comfortable and tight and re-connect PLUSMED pM-FM Full Face Mask.

8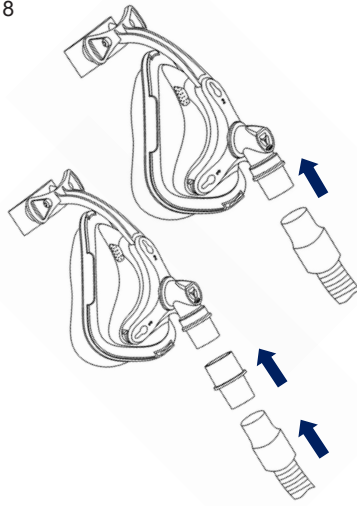

Connect the main air tubing to the neck or optional adjustable neck for easier taking out. Connect the free end of air tubing to your flow generator and open the air generator.


9


Adjust the main frame until the mask fits your face from all angles. The mask should be comfortable and its sealing be powerful. If necessary, adjust the head strap assembly.

Adjusting the Forehead Filling


1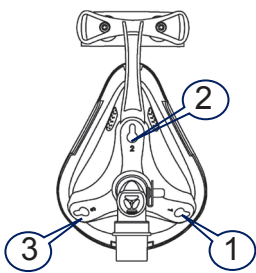

As shown in the illustration, the main frame has 3 different position to fit on different faces. Position "2" is the default position. To determine whether this default position is suitable for you, you can apply the facts determined in the 1st step in "Fitting the Mask" section.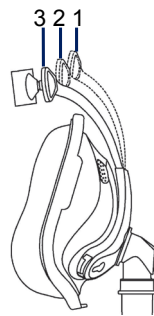


2

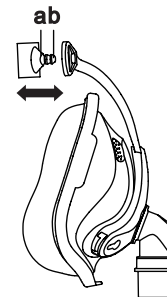
Set the forehead filling, by pushing or pulling the pawls in forehead filling. If it is comfortable, proceed to the 4th step in "Fitting the Mask" section.

3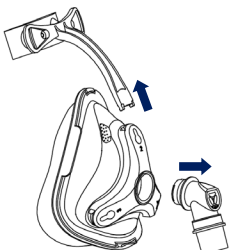


To confirm that the main frame can be separated from the buffer framework, take the bracket and pull out of forehead framework.

4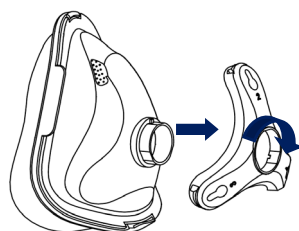


Separate the main frame and turn it to another direction.

5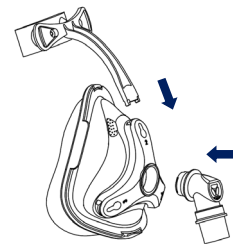


Place the forehead frame to adjustment position and fit the mask by inserting the bracket. Repeat the steps between 2 - 5 until the mask fits most comfortably and perfectly.


Disassembling the Mask into Parts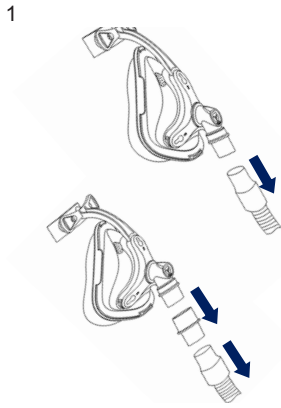


Remove the air tubing or on-demand tightening neck.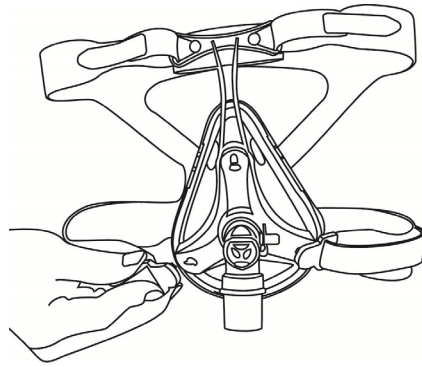


Open the clips and remove the upper belts of the head apparatus. It is not necessary to remove clips from the head apparatus.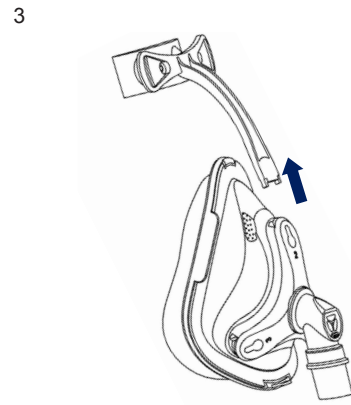


Put the forehead framework from the main frame.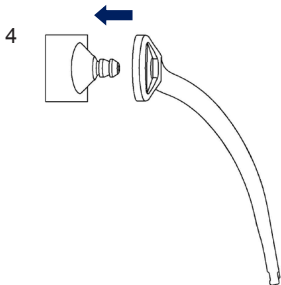

Remove the forehead filling from framework.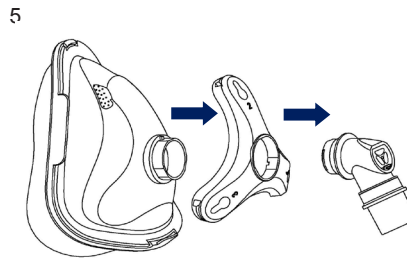

Remove the bracket group from the mask framework. The main framework will be separated.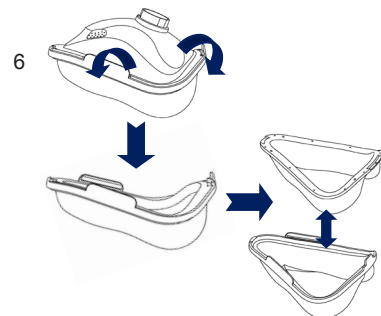

Remove thin filling and bumper from the bumper frame.

Cleaning the Mask at Home

Notes:

- Mask and head apparatus should be cleaned only by hand washing.
- The headgear may be washed without removing the clips.

⚠ CAUTIONS

- Do not use solutions containing bleach, chlorine, alcohol, aromatics, moisturizers, antibacterial agents, or scented oils to clean any part of the system or air tubing. These solutions may cause damage and reduce the life of the product.
- Exposure to direct sunlight or heat for any part of the system or tubing may cause deterioration.
- If there is a visible component breakdown (cracking, tearing or splitting etc.), this part should be disposed and changed with a new one.

Daily Use/Actions to be taken after Each Use

To ensure optimum mask leak proofing, facial oils on the buffer should be removed before each use. Clean the mask by lightly scrubbing in warm water (30°C) using a light soap or diluted dish-washing detergent.

If cover of raising element port needs cleaning, use a soft bristle brush.

Rinse all components with water in drinking water quality and leave them in a dry place where they shall not be exposed direct sunlight. Do not immerse the mask.


Weekly

Wash the head apparatus and all components in warm (approx. 30°C) and soapy water. Use pure soap. Rinse all components thoroughly and leave them in a dry place where they shall not be exposed direct sunlight.


Disinfection of the Mask during Use with Multiple Patients

PLUSMED pM-FM mask should be disinfected before using in various patients. Detailed disinfection instructions are available in Disinfection Instructions section and these instructions must be followed in multiple patient usage.


Re-Mounting the Mask Again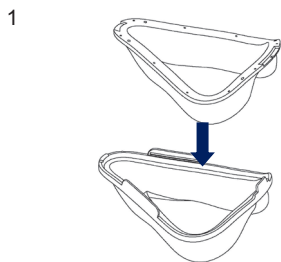


Align the buffer with thin filling.

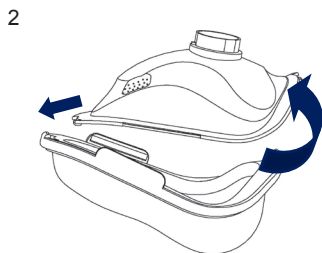
Close the buffer with thin filling (cover). Confirm that the thin filling and buffer face the same direction.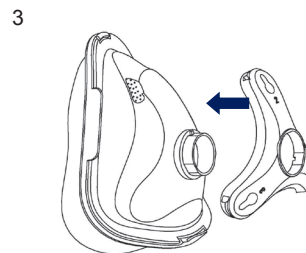


Press down slightly the main frame on the buffer framework. Make sure you feel comfortable in adjusted position and leak proofing is provided.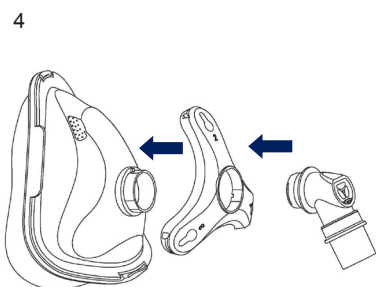


Connect the bracket group to the mask group. To do this, press the bracket fully in its place through the inside of leak-proofing ring.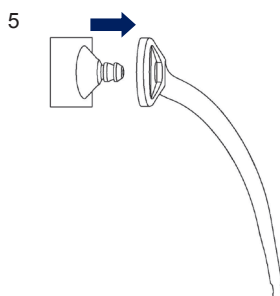


Place the forehead filling in the forehead frame to the desired position.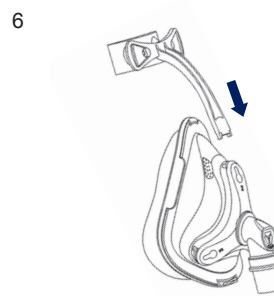

Place the forehead frame into the selected main frame position.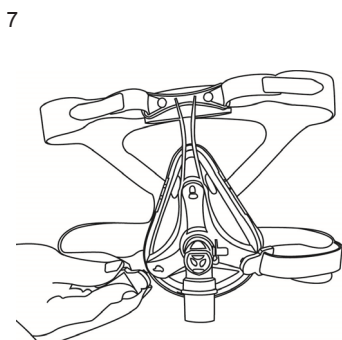

Attach the head apparatus by passing through holes and clips around the forehead frame.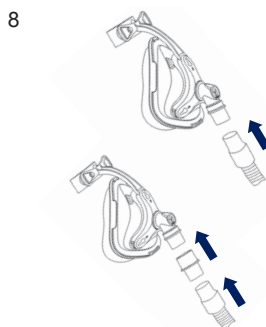


Connect the main air tubing to the neck or optional adjustable neck for easier taking out.

Troubleshooting

Problem	Possible cause	Possible solution
The mask does not have enough leak-proofing level or it is uncomfortable.	The mask may be worn or adjusted wrongly.	Follow carefully the instructions in "Fitting the Mask" section. Make sure the head apparatus not tightened very much. Adjust the main frame in a way to find a convenient position
	The mask length is wrong.	Consult the clinician.
	Thin filling is come loose around the buffer frame.	Check the correct placement of thin filling and relocate correctly according to instructions in "Remounting of the Mask" section.
	Thin filling of the mask is misaligned or creased on the face.	Make sure the thin filling is aligned according to the instructions in "Remounting of the Mask" section. Make sure thin filling has no wrinkling. Unbind the clips, check the buffer and thin filling and wear the mask on your face again.
	Buffer and thin filling of the mask may be dirty.	Clean the buffer and thin filling according to the instructions in "Cleaning of the Mask at Home" section.
The mask is too noisy.	Mask is incorrectly mounted.	Assemble the mask again.
	Ventilation holes are partially or completely clogged.	Clean ventilation holes according to the instructions in "Cleaning of the Mask at Home" section.

Technical Features

Pressure - Flow Curve

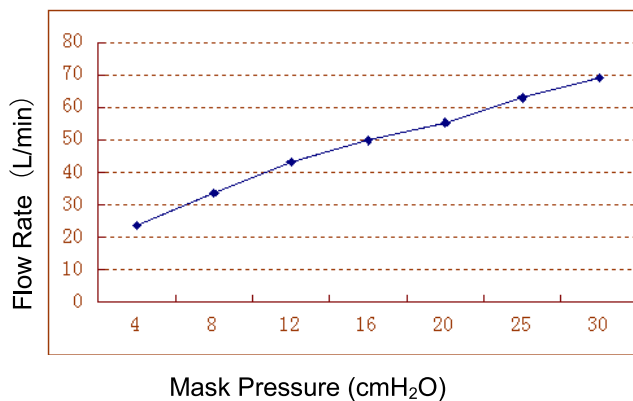Dead Space Information

Dead space is the empty volume of the mask up to the swivel. The dead space of the mask varies according to cushion sizes but is less than 246 ml.

Therapy Pressure

4 to 30 cmH₂O

Resistance

Drop in Pressure measured (average for 3 sizes)
 at 50 L/min: 0.1 cmH₂O
 at 100 L/min: 0.5 cmH₂O

Inspiratory and Expiratory Resistance	The inspiratory resistance (when used with the Non-Rebreathing Valve) 1.2 cmH ₂ O / L/S. The expiratory resistance (when used with the Non-Rebreathing Valve) 1.4 cmH ₂ O / L/S.
Sound	DECLARED DUAL-NUMBER NOISE EMISSION VALUES in accordance with ISO 4871. The A-weighted sound power level of the mask is 30 dBA, with uncertainty 3 dBA. The A-weighted sound pressure level of the mask at a distance of 1 m is 22 dBA, with uncertainty 3 dBA.
Environmental Conditions	Operating temperature: +5°C and +40°C Operating humidity: 0% and 93% relative humidity, without condensation Storage and transport temperature: -20°C and +55°C Storage and transport humidity: 0% and 93% relative humidity, without condensation

Preservation

Make sure the mask is clean and dry thoroughly before storing it for a long period of time. Store the mask in a dry place not exposed to direct sunlight.

Disposals

PLUSMED pM-FM does not contain any harmful substances and can be disposed together with normal household wastes.

Symbols

System and Packaging

Caution, consult accompanying documents

Lot number

Does not contain latex

Temperature limitation

Humidity limitation

User Manual

 This specifies a Warning or Alert status and it warns you against possible injury or explains measures required to be taken for secure and effective usage of the device.

Limited Warranty

We announce that PLUSMED pM-FM Full Face Mask, together with all components, frame, buffer, head apparatus, tubing and other accessories are under warranty for a period of ninety (90) days for material and workmanship flaws from the date of purchase.

To use your rights under this warranty, contact your local vendor and contact the manufacturer.

Trimpeks İth.İhr.Tur.ve Tic.A.Ş.

Eski Büyükdere Cad. Yunus Emre Sok.
Topçu İş Merkezi No:1/12,
34418, 4.Levent, İSTANBUL, TÜRKİYE
Tel +90 212 319 50 00
Fax +90 212 319 50 50

Trimpeks_IB_pM-FM_ver151601

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