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Specially designed for comfortable and convenient application of heat or cold heat or cold to painful areas of body

- Convenient pack and cover all in one
- Provides effective relief from muscular pain and helps reduce swelling
- Can be used in case of wide range of tissue injuries
- Reusable pack is perfect for continuous use at home

EN

Usage directions for cold use

1. Keep pack in freezer for a minimum of two hours. Store the pack in freezer when not use.
2. Put the pack into the pouch provided or towel cover and then apply to injured area
3. Apply for no longer than 20 min. and repeated every 1-1.5 hours if needed

Usage directions for hot use

Hot water method

1. Boil approximately 4-5 liters of water, and then remove water from heat.
2. Immerse the pack in water for approx. 10 min.
3. Remove the pack and put it into towel cover, then apply

Microwave Method

Microwave method because microwave ovens vary in power, you need to find the correct heating time carefully.

1. Before you begin, flatten the pack and ensure the gel is evenly distributed
2. If pack is at room temperature, expose on High for 1 min and let stand for 1 min. If more heat is necessary, try a further 10 secs at a time with equal standing until correct temperature is reached.
3. If frozen, expose on High initially for 75 secs then let stand for 75 secs. Further exposures should be in 10 secs increments, with equal standing time, until correct temperature is reached.
4. Record the total time for future use.

Caution

Do not apply directly to skin, always wrap pack in the pouch or towel cover. This product should not be used by people who are insensitive to temperature extremes, such as those with circulatory problems, diabetes, nerve damage or paralysis. Adult supervision is recommended if used by children. If the pack is punctured or opened and the gel is accidentally swallowed, drink plenty of water. If gel contacts eye, wash out thoroughly with water. Consult your doctor if you have concerns of your health. Check pack for leakage before use. Clean pack before use.

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